

- In case of use of any topical medications (on the penis or the vagina) please consult a doctor or a pharmacist before use.
- For additional lubrication you may use personal lubricants, designed for use with condoms or other water-based lubricants.
- Wash hands - as well as penis, vagina and surrounding areas - before and after sex and take special care to wash off lubricant after use of condom. This cuts the chance of infection.
- Avoid contact with eyes.
- This product should not be used if you or your partner have any inflammation or broken skin.
- Keep unused condoms in their packs in a cool, dry place away from direct sunlight (not in a wallet).
- If a new condom feels sticky or stiff or looks damaged in anyway or if the individual container is obviously damaged, throw it out - use a fresh one from an undamaged pack.

(\*Methaemoglobinemia risk stated in lay terms)

Mrs. Obels S.A, Bt, Général Waels, 53, 1030 Brussels, Belgium.  
Tel: +3227329954, Fax: +3227329003 Email: ms@obels.net www.obels.net

Manufactured by:  
HLL Lifecare Limited  
Peechookada, Thiruvananthapuram 695 005, India.  
e-mail: customerservice@moodsplanet.com.  
Visit us at www.moodsplanet.com  
Mfg.Lic.No. MFG/MD/2022/000666

ISO 4074 ③ For single use only.

- ⚠ Caution
- ☀ Keep away from sunlight
- ☞ Keep dry
- 📖 Consult instructions for use
- 🚫 Do not use if package is damaged

CE  
1639

**MOODS**  
CONDOMS

AW No. HLLPFT/LEGCC/5007/Rev.03/JAN 2024

LATEX



**INTENDED USE (Indication for use)**

MOODS Condoms, when used consistently and correctly, are effective means of protecting users and their partners to prevent pregnancy and sexually transmitted infections, although no contraceptive can guarantee 100% effectiveness.

**WARNINGS:**

Caution: This product contains natural rubber latex which may cause allergic reactions including anaphylactic shock. If you or your partner have had any reaction to latex rubber, do not use this product. If either partner develops a rash or irritation, stop use and consult a doctor.

- \* If either partner experiences respiratory difficulty or develops blue lips following use of this product, seek medical advice immediately and stop using the product.

**DIRECTION FOR CONDOM USE BEFORE SEX:**

Use a new condom every time you have sex – and before intimate contact (before foreplay, before penis gets anywhere near any body opening) to avoid exposure to any body fluid that can carry infection. Lesions, pre-ejaculate secretions, semen, vaginal secretions, saliva, urine and faeces can all transmit disease organisms.

1. Open the pack carefully so that the condom does not tear. Avoid contact with fingernails, jewellery etc for safety.

2. The condom is best worn on a fully erect, hardened penis. Pinch the nipple-end to expel any trapped air, and make space for the semen. Squeeze the tip of the condom and slowly insert the rim onto your hardened penis.
3. Continue squeezing the tip while slowly unrolling the condom. If the condom does not unroll, it is on wrong. Throw it away. Start over with a new one. Make sure that your condom fits snugly on your penis before any contact occurs between the penis and your partner's body, hold the ring of condom to prevent it from slipping off / tightening excessively to assist in the prevention of sexually transmitted diseases and pregnancy.

**AFTER SEX:**

After you have ejaculated, hold the rim of the condom and pull your penis out before it becomes flaccid. Turn and move completely away from the vagina before you let go off the condom. Each condom is for single use and if reused, there might be an increased risk of failure or infection. Dispose off the condom hygienically by wrapping it in tissue and dispose in a waste bin or bury it. If the condom breaks and semen spills or leaks out, do not panic, but wash semen away with soap and water. In case of condom leak/burst during use - Seek medical advice asap, at least within 72 hrs

- Never let a latex condom touch oil in any form - no petroleum jelly, no baby oil, no mineral oil, no vegetable oil, no massage oil - oil rots rubber. No body lotions, no butter no margarine, no mustard oil, etc. (i.e., no oil based lubricants) as these can damage the condom. Avoid contact with talcum / baby powder because it may contain oil.

AW No. HLLPFT/LEGCC/5007/Rev.03/JAN 2024

0249-HLL MOODS GCC insert  
80 mm (w) x 60 mm (h)