




CE Certified Male Condoms at HLL Lifecare Ltd

TECHNICAL SPECIFICATIONS & BRANDS



एचएलएल लाइफकेयर लिमिटेड
HLL Lifecare Limited

Sreejith



HLL Lifecare Limited (HLL) formerly known as Hindustan Latex Limited is a Mini Ratna Government of India Enterprise under the Ministry of Health & Family Welfare. HLL was established in 1966 for the manufacture and supply of condoms under the National Family Planning Programme of Govt. of India. HLL has traversed a long path, over the past fifty eight years. From a modest condom packing unit to a condom manufacturer, to an organisation manufacturing the widest range of contraceptives, HLL has emerged as one of the foremost providers of healthcare products and services in India. With a wider spectrum of activities, HLL touches the lives of millions across the world thereby rightly realising its motto of 'Innovating for Healthy Generations'.

Its extensive marketing infrastructure covers retail outlets, hospitals, medical professionals and villages across India. Globally, its brands are present in over 86 countries.

The core focus of HLL is to promote sexual health through high-quality male condoms designed to :

- Reduce the risk of sexually transmitted infections (STIs)
- Prevent unintended pregnancy.

The company's condoms are manufactured in accordance with International Standards and Regulations to ensure the highest quality and safety.

This technical brochure provides comprehensive information on the CE Certified Male Contraceptive condoms manufactured by HLL.

COMPANY AND REGULATORY INFORMATION

COMPANY NAME : HLL Lifecare Limited

ADDRESS : HLL Lifecare Limited
Peroorkada Factory, Peroorkada P.O
Thiruvananthapuram, Kerala, India. Pin: 695005

PHONE : +91 -471 235 4949

EMAIL : mrhll@lifecarehll.com

WEBSITE : www.lifecarehll.com

SRN : IN-MF-000022126

Notified Body Details

NAME : SGS Belgium NV

ADDRESS : SGS Belgium NV, SGS House
Noorderlaan 87 2030 Antwerp Belgium,
t +32 (0) 3 545-48-48 f +32(0)3 545-48-49

WEBSITE : www.sgs.com

NOTIFIED BODY NO : 1639

Authorized Representative

NAME : OBELIS S.A
ADDRESS : Bd. Général Wahis,53, 1030 Brussels, Belgium
WEBSITE : <https://www.obelis.net/>
NOTIFIED BODY NO : +3227325954
SRN : BE-AR-000000106

**MALE CONTRACEPTIVE
CONDOMS : Brands & Technical Data**

PRODUCT NAME : Natural Rubber Latex Condoms (Pre-lubricated)

BRAND/PROPRIETARY NAME : Moods, Protec, Love Brand

Basic UDI-DI : 89040685NLMCVN

EMDN CODE : U110101

MDA / MDN CODE : MDN 1210- Non-active non-implantable devices used for contraception or prevention of the transmission of sexually transmitted diseases

MDS & MDT CODE(S) : MDT 2004, MDT 2011

BRANDS / VARIANTS

Moods Regular




Moods Ultrathin



Moods Dotted



Moods Ribbed

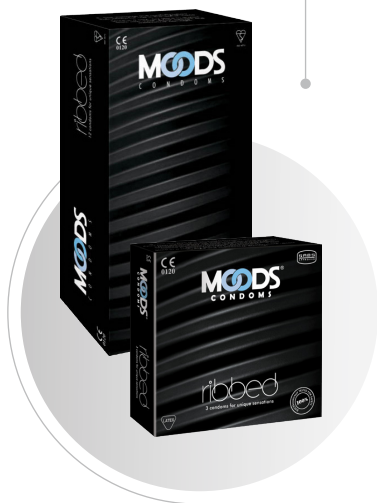

Moods Banana



Moods bubblegum



Moods Chocolate



Moods Coffee



Moods Strawberry




Moods Scented




Moods Coloured



Moods Extra Long

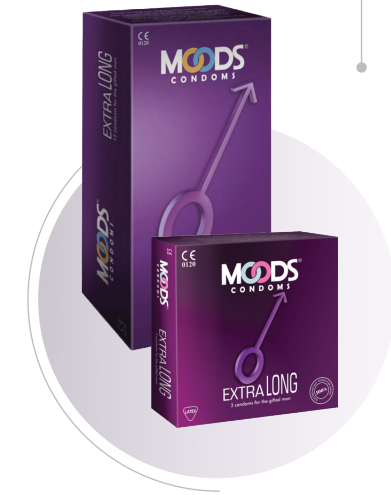


Moods Gold 1500 Dots

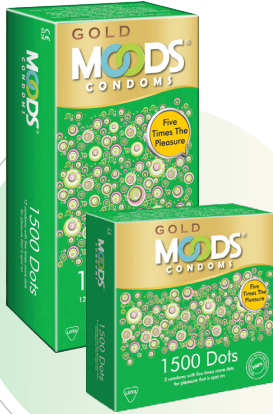

Moods Gold Blaze




Moods Gold Cool

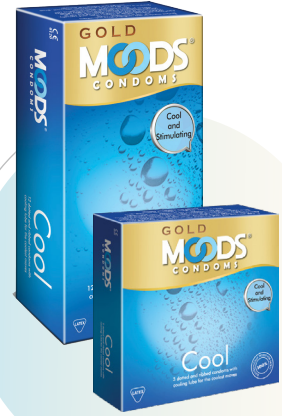


Moods Gold Joyride



Moods Grape

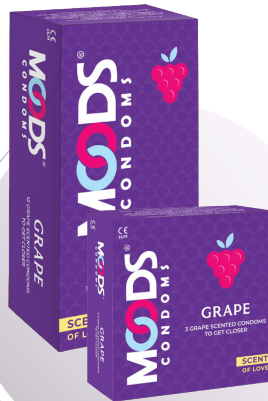

Moods Diamond

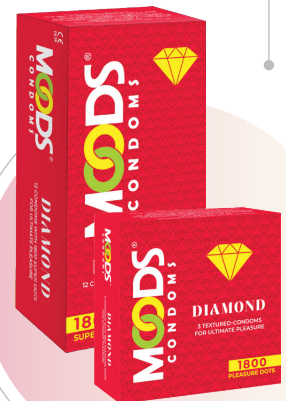

Moods 3X Fun



Protex Ultrathin



Protex Bubblegum



Protex Vanilla



Protex Strawberry



Love Brand



LIST OF BRANDS [CE MARKED]

PRODUCT NAME	CONDOM TYPE	COLOUR	FLAVOUR	FRAGRANCE	TEXTURE	LUBRICANT
Moods Regular	53 ± 2mm	Natural	Nil	Nil	Plain	Silicone oil
Moods Ultrathin	53 ± 2mm	Pink	Nil	Nil	Plain	Silicone oil
Moods Dotted	53 ± 2mm	Green	Nil	Nil	Dotted	Silicone oil
Moods Ribbed	53 ± 2mm	Violet	Nil	Nil	Ribbed	Silicone oil
Moods Banana	53 ± 2mm	Pink	Banana	Nil	Plain	Silicone oil
Moods Bubblegum	53 ± 2mm	Pink	Bubblegum	Nil	Plain	Silicone oil
Moods Choco	53 ± 2mm	Pink	Chocolate	Nil	Plain	Silicone oil
Moods Coffee	53 ± 2mm	Pink	Coffee	Nil	Plain	Silicone oil
Moods Strawberry	53 ± 2mm	Pink	Strawberry	Nil	Plain	Silicone oil
Moods Scented	53 ± 2mm	Pink	Nil	Rose	Plain	Silicone oil
				Musk	Plain	Silicone oil
				Jasmine	Plain	Silicone oil
Moods Coloured	53 ± 2mm	Green	Nil	Nil	Plain	Silicone oil
		Violet	Nil	Nil	Plain	Silicone oil
		Pink	Nil	Nil	Plain	Silicone oil
Moods Extralong	53 ± 2mm	Pink	Nil	Nil	1500 dots	Silicone oil
Moods Gold 1500 Dots	53 ± 2mm	Natural	Nil	Nil	3 in 1 condom	Silicone oil
Moods Gold Joyride	53 ± 2mm	Natural	Nil	Nil	3 in 1 condom	Cooling gel
Moods Gold Cool	53 ± 2mm	Natural	Nil	Nil	3 in 1 condom	Warming gel
Moods Gold Blaze	53 ± 2mm	Natural	Nil	Nil	Plain	Silicone oil
Moods Grape	53 ± 2mm	Purple	Grape	Nil	Dotted (Diamond Shape)	Silicone oil
Moods Diamond	53 ± 2mm	Purple	Nil	Nil	3 in 1 condom	Silicone oil
Moods 3X Fun	53 ± 2mm	Natural	Nil	Nil	Plain	Silicone oil
Protec Ultrathin [3s & Bulk]	53 ± 2mm	Pink	Nil	Nil	Plain	Silicone oil
Protec Bubblegum [3s & Bulk]	53 ± 2mm	Pink	Bubblegum	Nil	Plain	Silicone oil
Protec Vanilla [3s & Bulk]	53 ± 2mm	Pink	Vanilla	Nil	Plain	Silicone oil
Protec Strawberry	53 ± 2mm	Pink	Strawberry	Nil	Plain	Silicone oil
Love Brand	53 ± 2mm	Natural	Nil	Nil	Plain	Silicone oil



TECHNICAL SPECIFICATIONS, INTENDED USE & MEDICAL GUIDELINES

DEVICE NAME : NATURAL RUBBER LATEX CONDOMS (PRE-LUBRICATED)

DEVICE DESCRIPTION

HLL manufactured male contraceptive condoms are made of natural rubber latex. Male condoms consist of a cylindrical rubber sheath with one end open. The open end terminate in a rolled ring of latex, called an integral bead, 'rim' or 'ring'. The condoms have straight, parallel sides without constrictions and with a visible shoulder leading to a reservoir pouch at the closed end. It is a thin sheath that is placed over the erect penis before sexual intercourse to prevent unintended pregnancy and provide a barrier against sexually transmitted infections (STIs).

Lubricated condoms are packed in rolled condition in hermetically sealed individual containers and are capable of being unrolled readily. Condoms comes in different textures and colours. Silicone Oil is added to the condoms which act as the lubricant. Flavours/fragrances are also added along with the Silicone Oil. Condoms are made against the specifications stipulated as per ISO 4074:2015.

INTENDED USE

Intended use of HLL condom is to reduce the risk of transmission of sexually transmitted infections and to prevent pregnancy. HLL Male Condoms, when used consistently and correctly, are effective means of protecting users and their partners from unwanted pregnancy and sexually transmitted diseases including HIV that causes AIDS, although no contraceptive can guarantee 100% effectiveness.

INTENDED USE ENVIRONMENT

Condoms can be used in any environment. The usage environment of condoms encompasses a range of factors, including sexual behavior, cultural norms, accessibility, education, relationship dynamics, risk perception, and personal preferences. Understanding and addressing these factors are essential for promoting consistent and effective condom use as part of comprehensive sexual health initiatives.

MEDICAL INDICATIONS

Pregnancy Prevention

Male contraceptive condoms are proven to have a high effectiveness rate in preventing unintended pregnancy when used correctly and consistently.

STI Prevention

Condoms are proven to reduce the risk of sexually transmitted infections, including HIV/AIDS, gonorrhea, chlamydia, and syphilis, when used correctly and consistently.

INTENDED USER

Sexually active adult population (Adult male & partner)

TARGET PATIENT POPULATIONS

Sexually active adult population (Adult male & partner)

CONTRAINDICATION

The use of male contraceptive condoms may be contraindicated in individuals who have known allergies or hypersensitivity reactions to latex or any other materials used in the condoms. Individuals with lesions, sores, or ulcers in the genital area should also avoid using condoms as they may not effectively provide protection against STIs.

WARNINGS/CAUTIONS/PRECAUTIONS

Caution: This product contains natural rubber latex which may cause allergic reactions including anaphylactic shock. If you or your partners have had any reaction to latex rubber, do not use this product. If either partner develops a rash or irritation, stop use and consult a doctor.

If either partner experiences respiratory difficulty or develops blue lips following use of this product, seek medical advice immediately and stop using the product.

PRINCIPLE / MODE OF OPERATION

Male contraceptive condoms provide a physical barrier between the penis and the vagina, preventing the exchange of bodily fluids and interrupting the pathway of sperm to the egg. By blocking the direct contact between the penis and vagina, condoms reduce the chances of pregnancy and transmission of STIs. Also, condoms collect the semen during ejaculation.

COMBINATION WITH OTHER MEDICAL DEVICES

Condoms (Pre Lubricated) are in itself a medical device that serves its intended purpose of use without any combination with other devices or equipment. Condoms are not meant to be used in combination with oil based lubricants or other medicines. Condoms, however, can be used with Water Based Lubricants and the combination is safe and does not impair the specified performance of the product.

KEY PERFORMANCE & SAFETY CLAIMS

- PREGNANCY PREVENTION** : Male contraceptive condoms are proven to have a high effectiveness rate in preventing unintended pregnancy when used correctly and consistently.
- STI PREVENTION** : Condoms are proven to reduce the risk of sexually transmitted infections, including HIV/AIDS, gonorrhea, chlamydia, and syphilis, when used correctly and consistently.
- DURABILITY AND STRENGTH** : Condoms are proven to be durable and able to withstand normal use without breakage or tearing.
- COMFORT AND SENSITIVITY** : Condoms are proven to be comfortable to wear and not cause discomfort or decrease sexual sensitivity.
- EASY TO USE** : Condoms are easy to open, unroll, and apply correctly. Clear instructions for use are provided.
- SHELF LIFE** : Condoms are proven to have a specified shelf life of 5 years from the date of manufacturing and to maintain their effectiveness within that period if stored properly.

These claims and performance requirements are in accordance with the applicable standards, such as ISO 4074 and EU MDR 745/2017, which regulate the design, manufacturing, and performance of male contraceptive condoms



PLEASURE AND
PROTECTION



HLL Lifecare Limited
Peroorkada Factory
Peroorkada P.O
Thiruvananthapuram
Kerala, India. Pin: 695005
+91 -471 235 4949
mrhll@lifecarehll.com

www.lifecarehll.com